



902BTV-FM - Examination Stool Assembly Instructions

Parts Listing


A
Vinyl Upholstered
Seat with handle
1 Unit

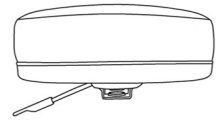


B
Gas Lift
1 Unit

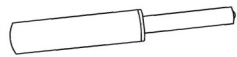

C
Steel Base
1 Unit

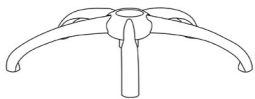

D
Casters
5 Units

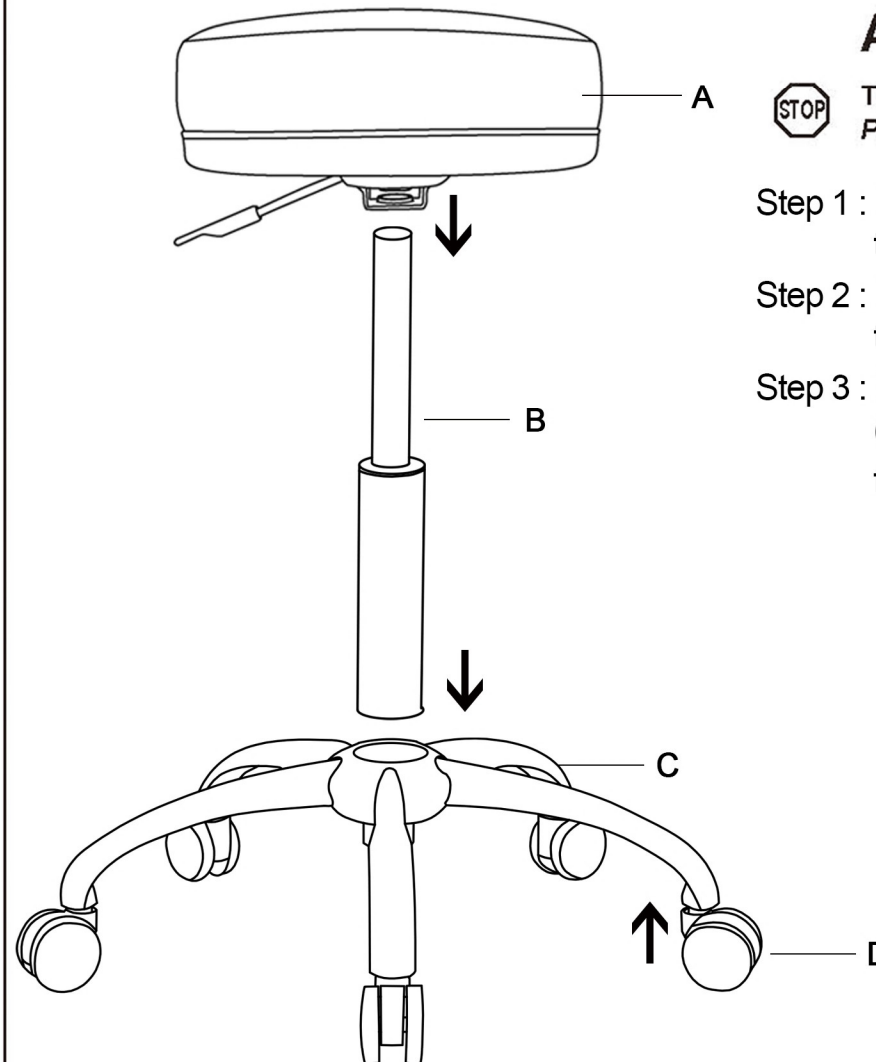

Assembly Instructions



Tools Needed: Phillips Head Screwdriver
Please read all instructions before assembly.

Step 1 : Insert the 5 Casters (D) into the outer holes on the Steel Base (C)

Step 2 : Insert the bottom end of the Gas Lift (B) into the center hole on the Base (C)

Step 3 : Place the Seat (A) on top of the Gas Lift (B)
(Make sure the top of Gas Lift has been properly fitted into the hole of Seat)

Weight Capacity : 250 lbs.

Assembly Notes:

During assembly, hand tighten screw only. When all screws are in place, you may then tighten all screws completely.

CAUTION:

1. Do not use this chair as a step ladder.
2. Check for loose screws and tighten them every 6 months.

Missing Parts? Assembly Questions?

Call us at 1-866-816-3822 or email help@regencyof.com for immediate assistance.